 **RingRescue**

RING RESCUE

REF RKM-EU, RKM-US, RKM-CA

FEI Number: 3015542886

Health Canada MDEL Number: 9115

U.S. Patent: 10,702,282

Ring Rescue Inc.
20 Orion Court
Dartmouth, Nova Scotia
Canada
1 (888) 640-7464
www.ringrescue.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in CANADA with domestic and imported materials

Contents

PAGE 4	EN
PAGE 5	AR
PAGE 6	BG
PAGE 7	CS
PAGE 8	DA
PAGE 9	DE
PAGE 10	EL
PAGE 11	ES
PAGE 12	FI
PAGE 13	FR
PAGE 14	HR
PAGE 15	HU
PAGE 16	IT
PAGE 17	KO
PAGE 18	NL
PAGE 19	PL
PAGE 20	PT
PAGE 21	RO
PAGE 22	SK
PAGE 23	SL
PAGE 24	SV

Indications for use

The device is indicated for use when a ring is stuck on a finger (ring entrapment), and removal is desired. The device, along with correct ring removal techniques, will often enable the removal of stuck rings.

Before use, watch tutorial:
www.ringrescue.com/tutorial

Background

Ring Rescue uses air pressure to temporarily compress a swollen finger, allowing for the removal of an entrapped finger ring. Ring removal techniques are then needed to remove the ring.

Intended patient population

Individuals 8 years and older.

Contraindications for use

Do not use in cases of severe finger injury.

Caution

Do not exceed 15 minutes of total compression time. Seek qualified medical advice in cases where finger bone fracture, nerve/vascular injury, skin laceration or damage, or injured ligaments and tendons are suspected.

Intended operators

Individuals that are reasonably expected to perform ring removal as part of their job.

Instructions for use

See the graphical step-by-step card that came with your device. We also see increased success with those who watch our instructional video at www.ringrescue.com/tutorial.

Cleaning & care instructions

Before and after each use, clean device and its components with 70% alcohol or 0.5% accelerated hydrogen peroxide wipes. Do not use any other cleaning substances or chemicals. Do not submerge any components in liquid. Keep device out of direct sunlight or ultraviolet light.

It is recommended a medical glove be applied over the affected hand during device use to preserve cleanliness.

Warranty

Two Year Limited Warranty

When operated and maintained according to all instructions supplied, if Ring Rescue fails due to a defect in material or workmanship within two years from the date of original retail purchase, the product will be repaired or replaced at the option of Ring Rescue Inc. for no additional charge. This warranty does not cover unsuccessful ring removal attempts. For full details see www.ringrescue.com/warranty.

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Copyright 2020, Ring Rescue Inc.